


# EO CHEX™

## ETHYLENE OXIDE STERILIZATION INDICATOR TAPE

### INSTRUCTION FOR USE

EO Chex™ Ethylene Oxide Indicator Tape is a single-use, beige pressure-sensitive tape imprinted with process indicator ethylene oxide reactive ink. EO Chex™ is suitable for securing sterilization packaging materials, including fabric or nonwoven wraps and pouches sterilization bags. Each roll is individually wrapped and features a 3-inch core diameter for use in standard tape dispensers.

To use, place tape on the package and press gently along the entire length to secure the pack. The pink lines on the tape are process indicators which turn orange-brown when exposed to ethylene oxide sterilization to distinguish between processed and unprocessed packs.

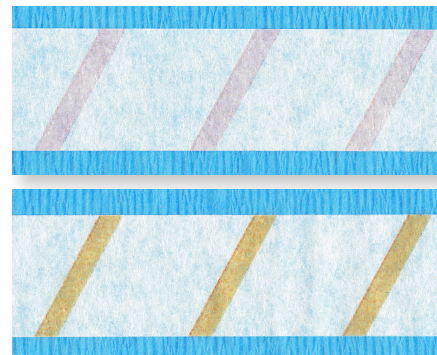

Reorder No.	Rolls/Box	Dimensions
26811500	48 Rolls	¾" x 55 YDS

*Pending 510(k), not available for sale within the United States.*

### STORAGE

EO Chex™ Ethylene Oxide Indicator Tape should be stored at room temperature (25°C / 77°F) in its original packaging. Relative Humidity is less than 60%. Keep EO Chex™ away from sunlight and chemicals, cleaning and disinfecting products.



36-04 Skillman Avenue  
Long Island City, NY 11101  
Tel. (718) 392-6650 • Fax. (718) 482-8909  
[www.proppermfg.com](http://www.proppermfg.com)

Rev. 04/24