



Instructions for Use: Propper Dust Covers for Sterility Maintenance

Product Description

Propper Sterility Maintenance Dust Covers are for use in the protection of wrapped packs, pouches, trays, or any other item from environmental contamination sources such as moisture, dust or lint during storage and transportation. They are clear plastic bags with an easy-open perforation which include a white, opaque area to write on. Propper Dust Covers available in a variety of sizes and in both self-sealing and heat sealing closures.

Indications for Use

Sterilized packages should be handled as little as possible between processing and sterile presentation. Sterility Maintenance Covers, also called Dust Covers, may be used to protect and extend the shelf life of properly packaged and sterilized items that could be subjected to environmental challenges or multiple handling before use. Only products specifically labeled as sterility maintenance covers or dust covers should be used for this purpose.

Instructions for Use

1. After processing, place cooled sterilized item into the Dust Cover.
2. Seal using a heat sealer designed for plastic-to-plastic sealing, use a similarly effective method of closure, or if your facility uses the self-seal product, remove the adhesive protective strip and fold the adhesive flap and press the self-adhesive strip to close the opening.
3. The lot or load control number and expiration statement should be visible through the sterility maintenance cover. Or the information should be included on the outside of the Dust Cover using a separate label or written with a non-toxic, soft-tipped marker designed for use on sterilization packaging, on the area provided on the Dust Cover.
4. During transportation of items enclosed in Dust Covers, proper care and handling should be observed always. Carrying the pack in both hands is the recommended practice.
5. To Open: Tear the seal at the perforation and remove the end of the Dust Cover completely for easy retrieval of the sterilized item.

Ordering Information

32101500	Dust Cover 10x15	Case	1,000
32102000	Dust Cover 12x20	Case	500
32102200	Dust Cover 16x22	Case	500
32103000	Dust Cover 16x30	Case	250
32103200	Dust Cover 22x30	Case	250
32121200	Self Seal Dust Cover 6x12	Case	1,000
32121500	Self Seal Dust Cover 10x15	Case	1,000
32122000	Self Seal Dust Cover 12x20	Case	500
32122200	Self Seal Dust Cover 16x22	Case	500
32123000	Self Seal Dust Cover 16x30	Case	250
32123200	Self Seal Dust Cover 22x30	Case	250