



STEAM CHEX™

AUTOClave INDICATOR TAPE

INSTRUCTION FOR USE

Steam Chex Blue Autoclave Indicator Tape is a pressure-sensitive tape imprinted with process indicator steam reactive ink. Steam Chex Blue Autoclave Tape is suitable for securing sterilization packaging materials, including fabric or nonwoven wraps, pouches and paper sterilization bags.

To use, place tape on the package and press gently along the entire length to secure the pack. The light green lines on the tape are process indicators which turn dark (black or dark brown) when exposed to steam sterilization to distinguish between processed and unprocessed packs.

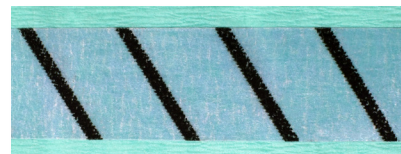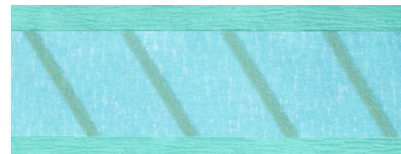

Reorder No.	Rolls/Box	Dimensions
26810800	48 Rolls	¾" x 60 YDS
26810900	36 Rolls	1" x 60 YDS

STORAGE

Steam Chex Autoclave Indicator Tape should be stored at room temperature (15°-30°C / 59°-86°F) in its original packaging. Relative Humidity should be below 50%. Keep Steam Chex away from sunlight and chemicals, cleaning and disinfecting products.



36-04 Skillman Avenue
Long Island City, NY 11101
Tel. (718) 392-6650 • Fax. (718) 482-8909
www.proppermfg.com