


STRATE-LINE®

AUTOCLAVE INDICATOR TAPE

INSTRUCTION FOR USE

Strate-Line Autoclave Indicator Tape is an all-purpose pressure-sensitive tape imprinted with process indicator steam reactive ink. Strate-Line tape is suitable for securing sterilization packaging materials, including fabric or nonwoven wraps, pouches and paper sterilization bags.

To use, place tape on the package and press gently along the entire length to secure the tape to the pack. The white lines on the tape are process indicators which turn dark (black or dark brown) when exposed to steam sterilization to distinguish between processed and unprocessed packs.

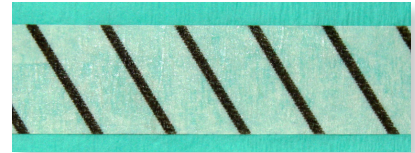

Reorder No.	Rolls/Box	Dimensions
26800500	48 Rolls	¾" x 60 YDS
26800600	36 Rolls	1" x 60 YDS

STORAGE

Strate-Line Autoclave Indicator Tape should be stored at room temperature (15°-30°C / 59°-86°F) at less than 50% relative humidity). The tape should not be exposed to light. Keep away from any chemicals, cleaning or disinfecting products.



36-04 Skillman Avenue
Long Island City, NY 11101
Tel. (718) 392-6650 • Fax. (718) 482-8909
www.propermfg.com