

## Strate-Line® Autoclave Indicator Tape

### TYPE 1 PROCESS INDICATOR FOR STEAM STERILIZATION

#### Technical Data Sheet

- Strate-Line Autoclave Indicator Tape is designed to seal packs that are exposed to steam sterilization. The tape distinguishes between processed and unprocessed items in both gravity and pre-vacuum steam sterilization processes.

#### gravity sterilizers\* operating at

121°C for 30 minutes  
 132°C for 3 minutes  
 132°C for 10 minutes  
 132°C for 15 minutes  
 135°C for 3 minutes  
 135°C for 10 minutes

#### pre-vacuum sterilizers\* operating at

132°C for 3 minutes  
 132°C for 4 minutes  
 134°C for 3 minutes  
 134°C for 4 minutes  
 135°C for 3 minutes



- Designed to use on disposable wraps, pouches and sterilization bags. When applying the tape, apply pressure to ensure that the tape will stay in place throughout the steam autoclave process. After use, the tape can be easily removed.
- Made of a saturated crepe paper printed with non-lead based ink, the white/beige indicator lines turn dark brown/black once exposed to the steam process. It is coated with a synthetic, solvent-free adhesive.
- This product is not made with natural rubber latex.
- Lead is not used in the manufacturing of the ink of indicator lines.

Strate-Line Autoclave Indicator Tape should be stored at room temperature (15°-30°C / 59°-86°F) at less than 50% relative humidity. The tape should not be exposed to light. Keep away from any chemicals, cleaning or disinfecting products.

#### Interpretation

Tape is a single use device that is interpreted after processing to distinguish processed and unprocessed package. Indicator lines on the tape turn dark brown/black when exposed to proper steam sterilization conditions.

Technical Data	
<b>Color</b>	Natural with printed beige indicator lines
<b>Backing</b>	Saturated crepe paper
<b>Adhesive</b>	Synthetic, solvent-free adhesive
<b>Indicator</b>	Non lead based ink
<b>Thickness</b>	5.5 mil (0.14mm)
<b>Adhesion to Steel</b>	70 oz/in (19N/25mm)
<b>Tensile Strength</b>	20 lb/in (87N/25mm)
<b>Elongation</b>	7%
<b>Meets International Standard</b>	ISO 11140-1:2014
<b>SKUs / Size</b>	26800500 / 3/4" x 60 yds 26800600 / 1" x 60 yds

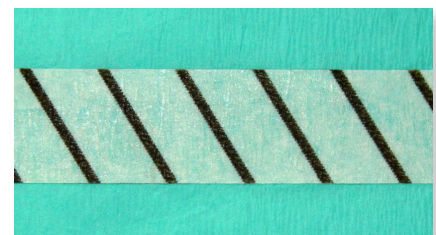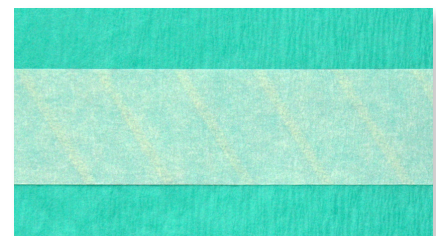

\*Always consult instructions from manufacturers of sterilization units concerning the recommended sterilization cycle.

Professional • Responsive • On Time • Proactive • Proven • Efficient • Respectful