

## Verline™ Autoclave Indicator Tape

### TYPE 1 PROCESS INDICATOR FOR STEAM STERILIZATION

#### Technical Data Sheet

- Verline Autoclave Indicator Tape is designed to seal packs that are exposed to steam sterilization. The tape distinguishes between items processed and unprocessed in both gravity displacement and pre-vacuum steam sterilization processes.
- Designed for use in gravity sterilizers operating at 121°C (+3°C) for 30 minutes or pre-vacuum sterilizers operating at 132°C (+3°C) for 4 minutes and 135°C (+3°C) for 3 minutes.
- Verline is designed to use on disposable sterilization wraps, like Propper Steri-Wrap and Steri-Wrap Unite. Verline is pressure sensitive tape, use pressure when applying the tape to ensure that adhesive attaches to the wrap. After use, the tape removes easily.
- Made of a saturated crepe paper printed with non-lead based ink, the green indicator lines turn dark brown or black once exposed to the steam process. It is coated with a synthetic, solvent-free adhesive.
- Verline is not made with natural rubber latex.
- Lead is not used in the manufacturing of the ink indicator lines.

Verline Autoclave Indicator Tape should be stored at room temperature (15°-30°C / 59°-86°F) in its original packaging. Relative Humidity should be below 50%. Keep Verline away from sunlight and chemicals, cleaning and disinfecting products.

The shelf life of this product is 36 months from the date of manufacture, when stored at the recommended conditions.

#### Interpretation

Tape is a single use device that is interpreted for appropriate signal color after processing. Indicator lines on the tape turn dark brown or black when exposed to steam sterilization conditions.

Technical Data	
<b>Color</b>	Natural with printed green indicator lines
<b>Backing</b>	Saturated crepe paper
<b>Adhesive</b>	Synthetic, solvent-free adhesive
<b>Indicator</b>	Non lead based ink
<b>Thickness</b>	5.5 mil (0.14mm)
<b>Adhesion to Steel</b>	70 oz/in (19N/25mm)
<b>Tensile Strength</b>	20 lb/in (87N/25mm)
<b>Elongation</b>	7%
<b>Meets International Standard</b>	ISO 11140-1:2014
<b>SKUs / Size</b>	26810500 / 3/4" x 60 yds 26810600 / 1" x 60 yds

