



**propper**  
Manufacturing Co., Inc.

36-04 Skillman Avenue  
Long Island City, NY 11101  
Tel (800) 832- 4300  
Fax (718) 482-8909

[www.propermfg.com](http://www.propermfg.com)

## How to Use Gas-Chex<sup>®</sup> EO Sterilization Indicators

(For use in Ethylene Oxide Gas Sterilizers)

Reorder #: 269002 – Gas-Chex Class 5 Indicator Strips, 8 inch

Place a Gas-Chex Strip into each pack. After processing, when sterile presentation is made examine the Gas-Chex mark and the diamond mark at the opposite end. Both indicator ink marks must be **green**. Brown color or the original red unchanged indicate underprocessing, do not use the contents of the pack. Return for reprocessing and determine the cause of the failure.

Remember your signal of safety is **green**.