

Smalstrip® Sterilization Indicator Strips

Indications for Use:

For use in small packages. In large packages use Strate-Line® Sterilization Indicator Strips. Propper Smalstrip Sterilization Indicator Strips can be used in all hospital steam autoclaves operating at 250°F (121°C) or 273°F (134°C). When used as directed, Smalstrip indicators give visible proof that saturated steam has fully penetrated the package. After exposure, a uniform black line must appear along the entire length of the strip.

Instructions for Use:

Place a strip into each pack. The left end should be at the location considered to be the hardest for the steam to reach. The other end may project from the contents. After processing, when sterile presentation is made, examine the end projecting from the contents.

The indicator must be black. Pull the strip out of the contents and examine the entire line.

It must be black. Brown color indicates underprocessing; do not use the pack. Return for processing and determine the cause of the failure. Gray color may be indication of overprocessing or wet steam. It is best to correct these conditions.

Ordering Information:

Reorder no. 26510200

Quantity: 1 box of 250 strips

