



propper
Manufacturing Co., Inc.

36-04 Skillman Avenue
Long Island City, NY 11101
Tel (800) 832- 4300
Fax (718) 482-8909

www.propermfg.com

Steam-Dot Blu® Instructions for Use

For use in gravity displacement and pre-vacuum cycles

1. Affix the Steam•Dot Blu® Process Indicator to the outside of the sterilization container, cassette, pouch or package to be sterilized.
2. Run a normal steam cycle.
3. The initial blue color will turn pink, indicating that the item has been processed.
4. Color change does not guarantee that adequate sterilization conditions have been met.

Thanks,

Kelly House
Marketing Director
917-617-5372
khouse@propermfg.com

