



**propper**  
Manufacturing Co., Inc.

36-04 Skillman Avenue  
Long Island City, NY 11101  
Tel (800) 832-4300  
Fax (718) 482-8909

[www.propermfg.com](http://www.propermfg.com)

## Propper Twindicator® Strip Instructions for Use

Propper Twindicator Sterilization Indicator Strips can be used either as an 8-inch strip or, when torn in half, as a 4-inch strip. The white indicator line changing to black gives visible proof that saturated steam has fully penetrated the pack or pouch within which the strip was placed.

### How to use Twindicator® as an 8-inch strip:

One end should be at the location considered to be the hardest for steam to reach. The other end may project from the contents to give indication that the contents have been processed. The entire indicator line must be black. Brown color indicates under processing, and the contents must be reprocessed. Gray color may be indication of over processing or wet steam. Faulty sterilizer or cycle must be correct before reprocessing.

### How to use Twindicator® as a 4-inch strip:

Place the entire strip within pack or pouch and follow above instruction for determining that saturated steam has fully penetrated the pack.