

Magnalume™

Laryngoscope Disposable Blades

INTENDED USE

Propper disposable laryngoscopes are medical devices used by healthcare professionals to examine and visualize the larynx and facilitate tracheal intubation.

SPECIFICATIONS

Compatible with ISO 7376 (green code) light-emitting handles. Non-sterile.

WARNINGS

- Only trained personnel shall use a laryngoscope for intubation.
- Reuse of laryngoscope blade intended for single use has associated dangers of cross-patient infections and malfunction.

INSTRUCTIONS FOR USE

- Select laryngoscope blade type and size appropriate for the patient.
- Check to see that handle bulb functions correctly.
- Attach selected blade onto properly operational handle.
- Verify the blade is lit properly.
- Examine / intubate trachea following current accepted medical guidelines.

SPECIFICATIONS

Do not sterilize, reprocess, or re-use. Blades are contaminated after intubation, and it is the user's responsibility to ensure that all used blades are disposed in accordance with medical waste disposal guidelines.

Store in a clean, dry place at room temperature.
Discard after expiration date.



Single Use



36-04 Skillman Avenue
Long Island City, NY 11101
Tel. (718) 392-6650 • Fax. (718) 482-8909
www.proppermfg.com

Rev. 04/26